



LIYA SHAPE III

User Manual

Dear client:

Thank you for choosing our company's product.

This manual provides comprehensive instructions on the safe and effective use of [LS15]. The purpose of this manual is to familiarize users with the machine's features and functions, as well as the proper operating procedures to ensure safe and optimal performance. **This machine is intended to be operated only by personnel who are trained and qualified.**

- ◆ Before operating the machine, it is crucial to read the entire manual thoroughly.
- ◆ Follow the instructions provided in the manual to ensure proper installation and operation of the machine.
- ◆ The configuration will determine the appearance of the machine you receive.
- ◆ The user manual is applicable to all configurations of the machine, regardless of appearance.
- ◆ The information in this manual is subject to change without notice at the discretion of Winkonlaser.


Feel free to contact us if there are any problems during operating

Contents


Machine Components	1
Maintenance	2
Treatments	3
Contraindications	4
RotatingSlim A.....	5
RotatingSlim B.....	6
Quantum Focus RF	7
RF Magic Finger.....	7
EMS Magic Finger	9
Applicator Movement Guide.....	10
Parameters	11
Specification.....	13
Contact.....	14

LS15 is a cutting-edge device for both face and body treatments. Combining rotational and vacuum massage with radio frequency, electromagnetic waves, and red & blue light, it targets all skin layers to reduce localized fat, minimize cellulite, and improve skin firmness and circulation.

This manual provides a detailed description of the machine's components and functions, as well as the operational procedures to be followed to ensure safe and effective use. A list of the machine's components and their functions is provided in next section: Machine Components.

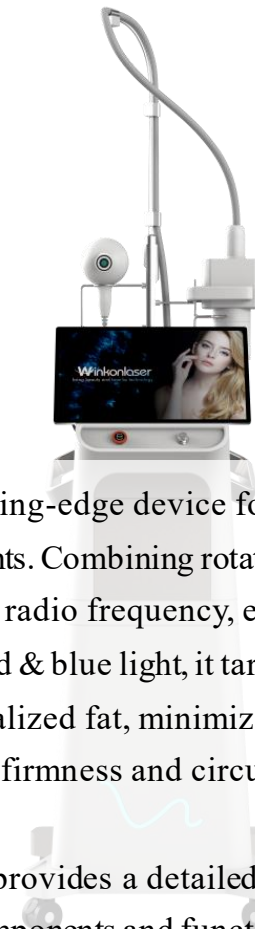

Machine Components

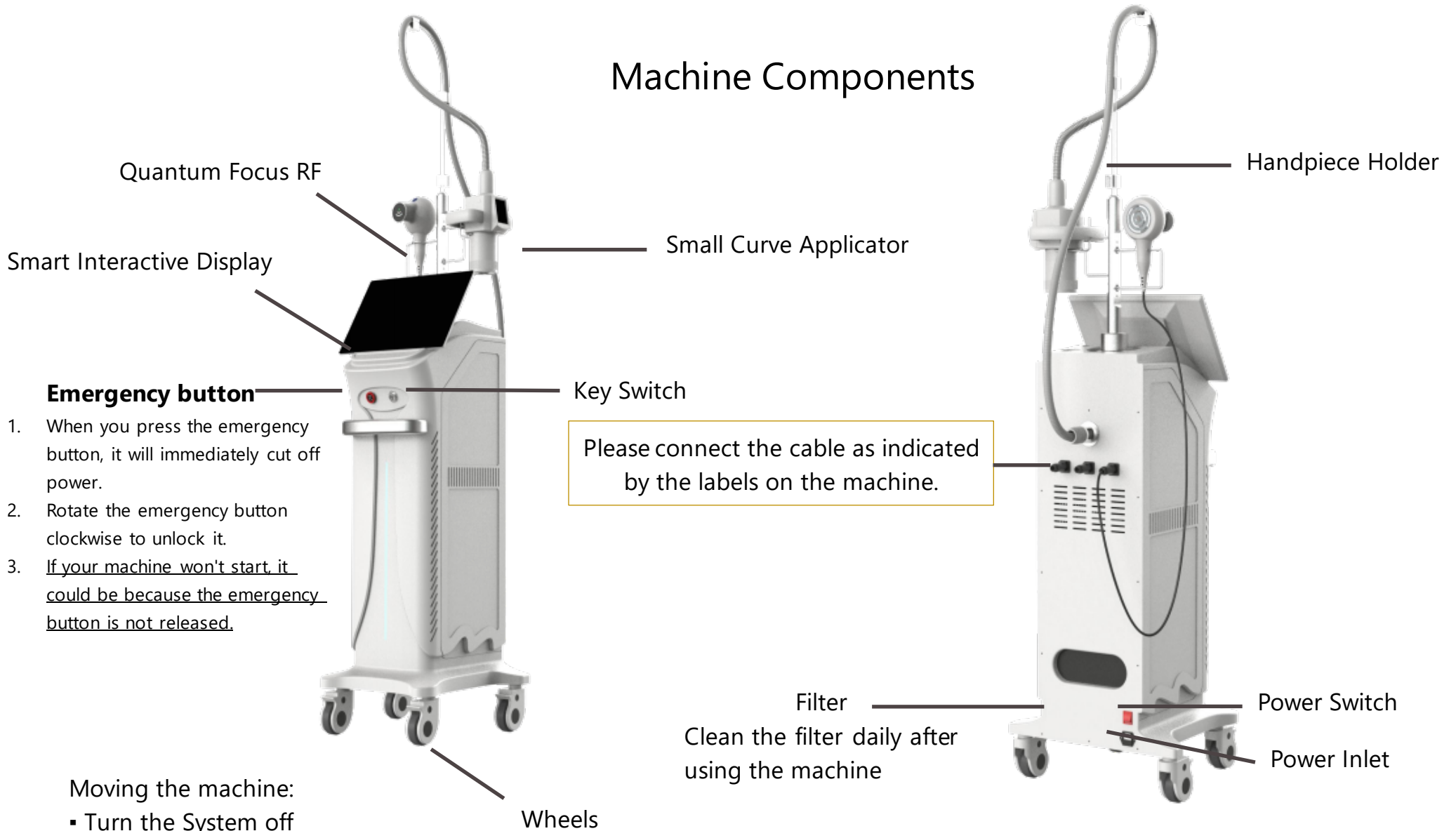




Maintenance

1. **Clean the filter** daily after using the machine. Use a screwdriver to remove the back cover, take out the filter, clean it thoroughly, and reassemble the parts before the next use.
2. Do not immerse any part of the system.
3. System Disposal: To comply with European Commission Directive 2002/96/EC on Waste Electrical and Electronic Equipment (WEEE) and other country/state's regulations, please DO NOT dispose any parts of this equipment in any location other than designated locations.
4. Use the original power cable to supply a stable power.
5. Store the machine in a cool, dry place, away from direct sunlight and extreme temperatures (heating, for example).
6. Take the plug off when not operating the machine for a long period of time.

Caution: Do not cross-connect cables of Magic Finger or use mismatched ports.



Correct

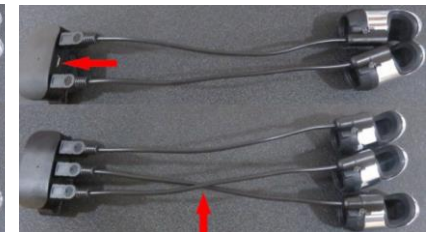

Incorrect


Treatments

RotatingSlim A

- Cellulite Reduction
- Fat Elimination
- Collagen Stimulation
- Vacuum Lymphatic Drainage
- Wavy Massage

RotatingSlim B

- Skin Tightening
- Wrinkle Reduction
- Puffiness Removal
- Facial & Body Application

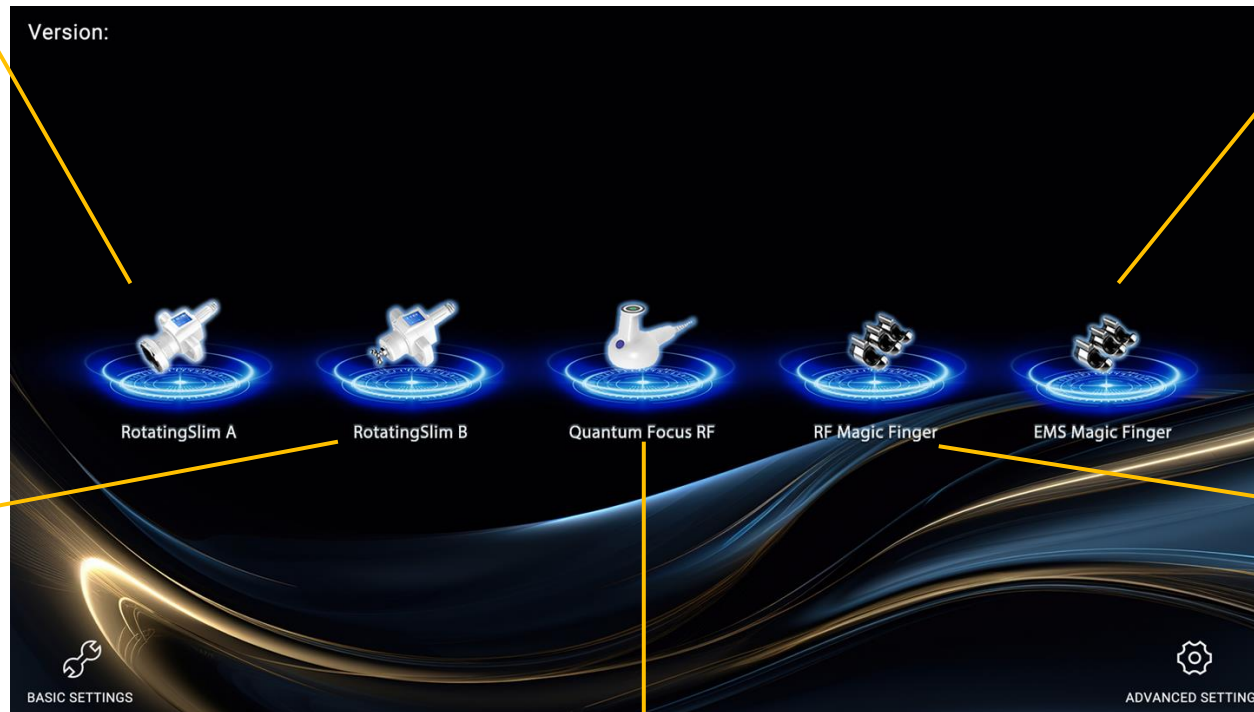

EMS Magic Finger

- Massage the skin, encouraging circulation and relaxation of muscles

RF Magic Finger

- Enhance skin tone and firmness

Quantum Focus RF

- Deep Tissue Healing
- LED Light Therapy
- Infrared Penetration
- Collagen Stimulation
- Skin Contouring

Contraindications

The following individuals should not use this product or undergo this procedure:

- Patients with heart conditions, implanted pacemakers, or hypertension.
- Individuals with acute inflammation, deep vein thrombosis, goiter, asthma, cancer, or similar medical conditions.
- Those experiencing vascular rupture, bleeding wounds, or open injuries.
- Pregnant women.
- Individuals who have undergone cosmetic surgery or have areas with artificial fillers.
- People with metal implants or materials in their body (including metal dental work).
- Individuals with inflamed skin or significant body swelling.
- Those with abnormal immune system function.
- Individuals with numbness or an inability to sense heat.

Caution: This machine is not suitable for clients with a BMI of 40 or higher.

RotatingSlim A



1. Use massage oil on the skin.
2. Start the vacuum level at a low setting and gradually increase it.
3. Do not use on wounds or scars.
4. Choose the suitable treatment probe based on the size of the area.

This refers to the duration of the gap between each vacuum cycle on the skin. If using the preset settings, adjustment is not required.

Duration of the treatment


Radio Frequency Intensity: Start at a low level for safety.

Vacuum Intensity: Use caution with clients who have a high BMI.

Vacuum Working Pattern

Provides preset settings for suction and release durations. Mode 6 maintains continuous vacuum on the skin.

This refers to how long the vacuum works on the skin. If using the preset settings, adjustment is not required.

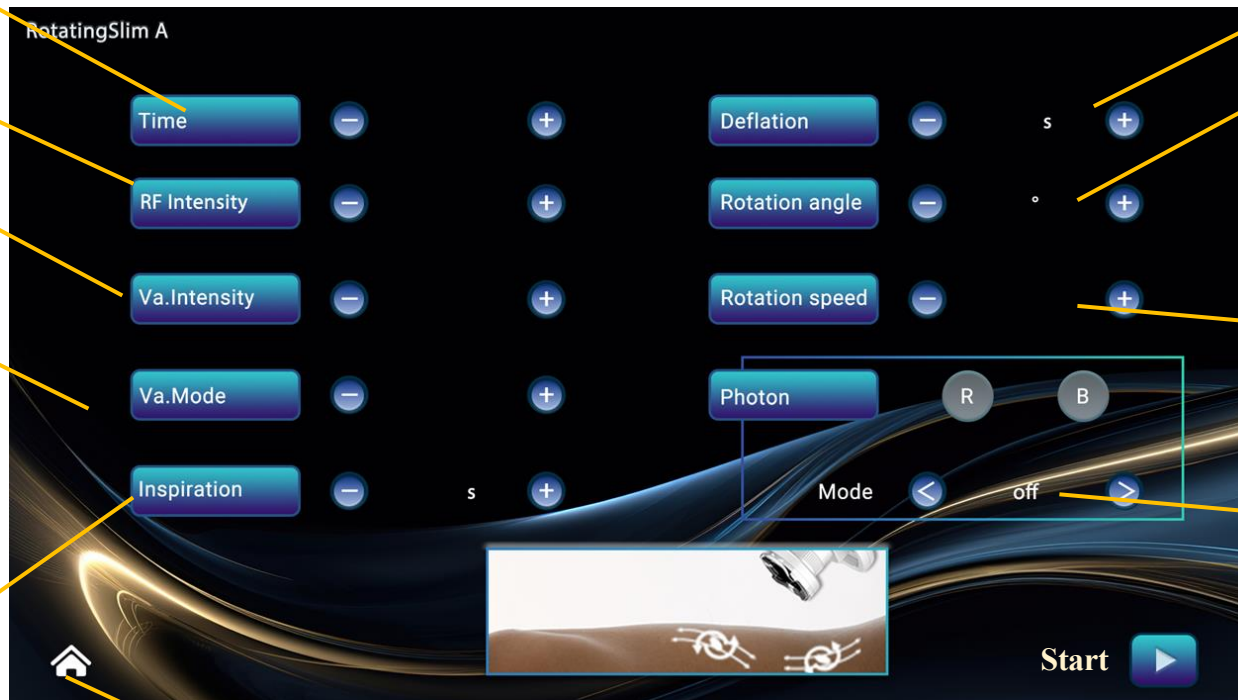

Rotation Angle: Defines the range of each probe rotation. Use with caution for patients with a high BMI.

Rotation Speed: Indicates how fast the rotation occurs.

Tap here to turn on or off the PDT light. R represents red light, and B represents blue light.

Return to the Home Page

RotatingSlim B



1. Use massage oil on the skin.
2. Start the RF intensity level at a low setting and gradually increase it.
3. Do not use on wounds or scars.
4. Choose the suitable treatment probe based on the size of the area.
5. When using the applicator on the skin, ensure all probes are in full contact with the skin, and **do not** remain in the same spot for more than 2 seconds.

The small massage head is suitable for the arms or calves.
The large massage head is ideal for the thighs or abdomen.



Duration of the treatment: 15–20 minutes per area is recommended.


Radio Frequency Intensity: Start at a low level for safety.

Rotation Speed: Indicates how fast the rotation occurs.

Tap here to turn on or off the PDT light. R represents red light, and B represents blue light.

Quantum Focus RF

1. Before the treatment, check if the client is wearing any electronic or metal items and have them removed.
2. Use massage oil on the skin.
2. Start the RF intensity level at a low setting and gradually increase it.
3. Do not use on wounds or scars.
4. Choose the suitable treatment probe based on the size of the area.
5. When using the applicator on the skin, ensure applicator is in full contact with the skin, and **do not** remain in the same spot for more than 2 seconds.

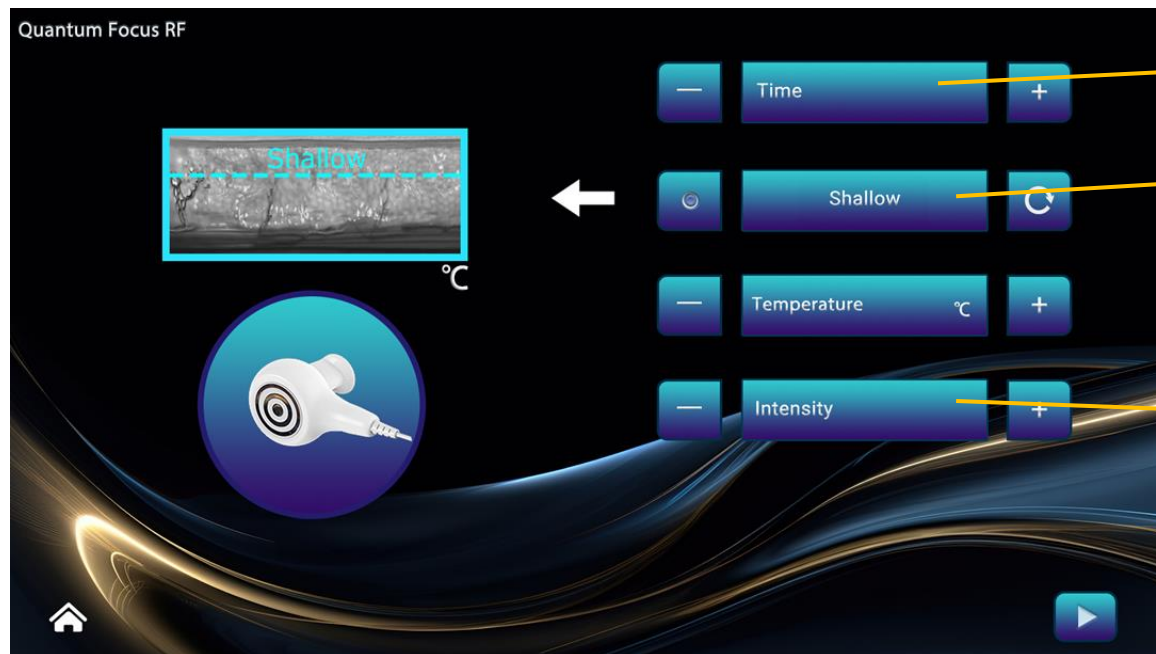

Duration of the treatment

Tap here to adjust the working depth of Radio Frequency. Select "Deep" or "Shallow" on the screen.


Vacuum Intensity: Use caution with clients who have a high BMI.

RF Magic Finger



1. Avoid wearing any metal ornaments, whether you are the operator or the patient.
2. Apply conductive gel or massage oil to the targeted area. Ensure the Magic Finger maintains full contact with the skin.
3. Do not remain in the same spot for more than 2 seconds during treatment.
4. Lower the intensity level when treating sensitive areas, such as the forehead or around the eyes.
5. Do not point the Q-IR light directly at the eyes.

Select the treatment area

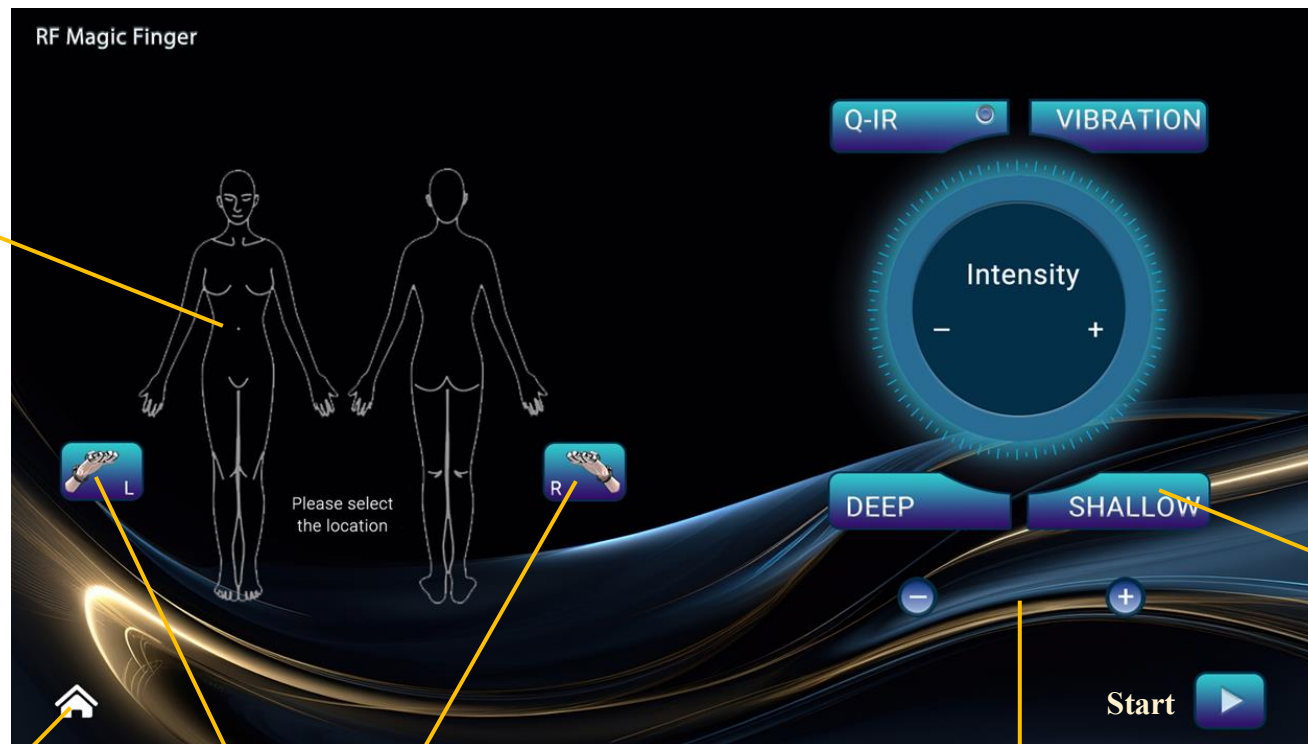

Return to the Home Page


Tap here to activate the left hand or right hand. Both can be used simultaneously.

This displays the countdown of the treatment time. Each area requires 10–20 minutes.


Tap here to adjust the working depth of RF. For facial treatments, select 'Shallow.' For areas with thicker tissue, such as the abdomen or thighs, select 'Deep.'

EMS Magic Finger

1. Avoid wearing any metal ornaments, whether you are the operator or the patient.
2. Apply conductive gel or massage oil to the targeted area. Ensure the Magic Finger maintains full contact with the skin.
3. Do not remain in the same spot for more than 2 seconds during treatment.
4. Lower the intensity level when treating sensitive areas, such as the forehead or around the eyes.
5. Do not point the Q-IR light directly at the eyes.

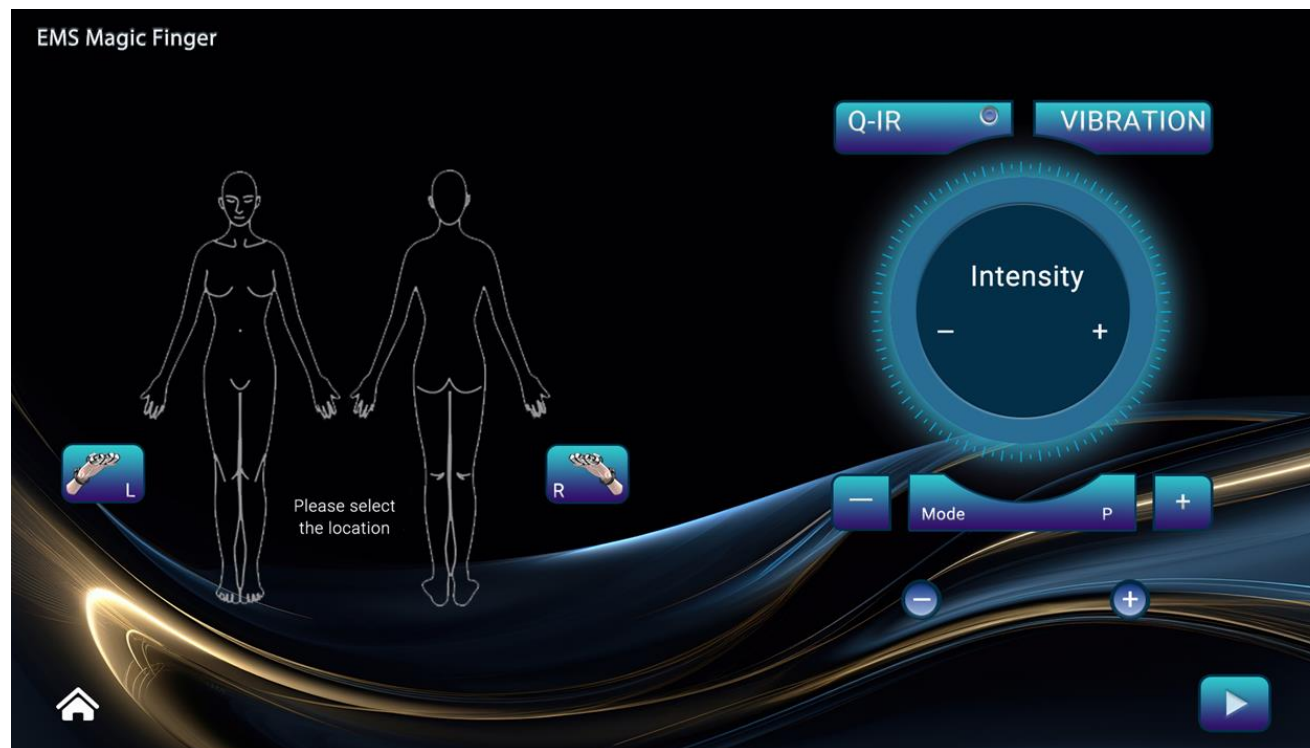

Applicator Movement Guide

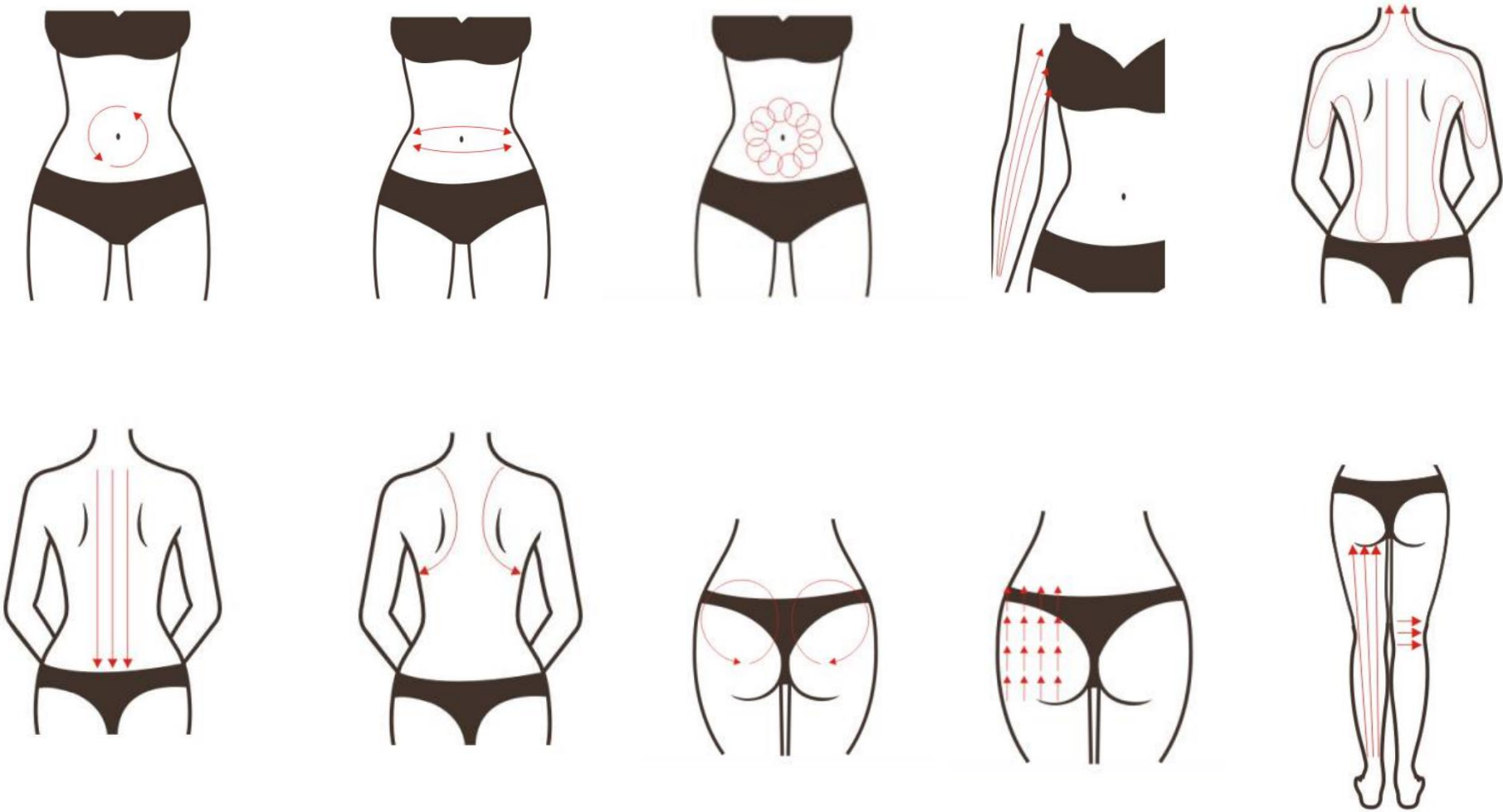


Parameters

15–20 minutes per area

Start with level 3


Start at 20, typically kept below 50

M1 is suitable for most clients. M2 to M5 gradually increase in intensity, with M6 providing continuous vacuum.

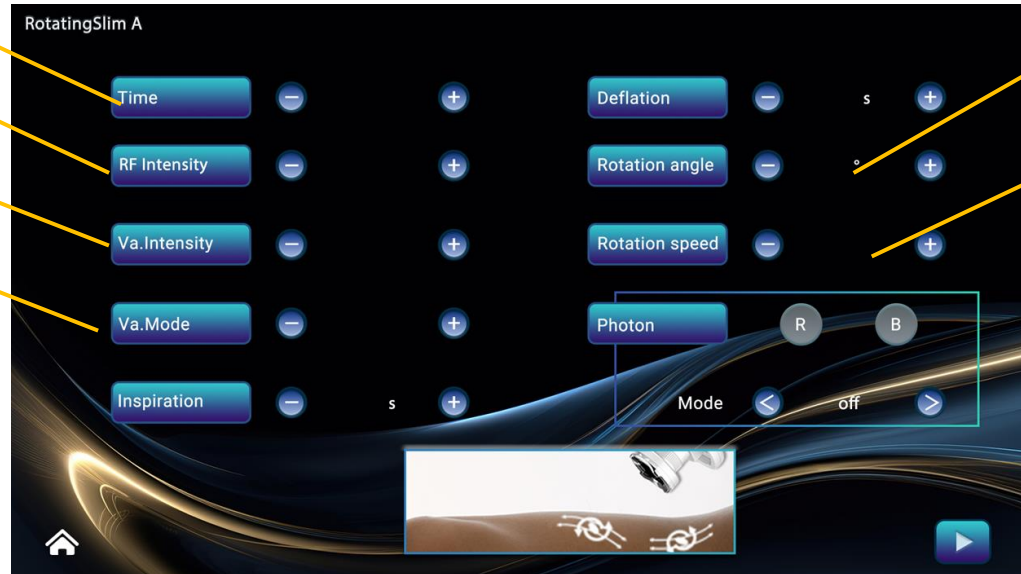

Set between 40–60. A higher angle will exert more pulling on the skin


Level 3 as the default

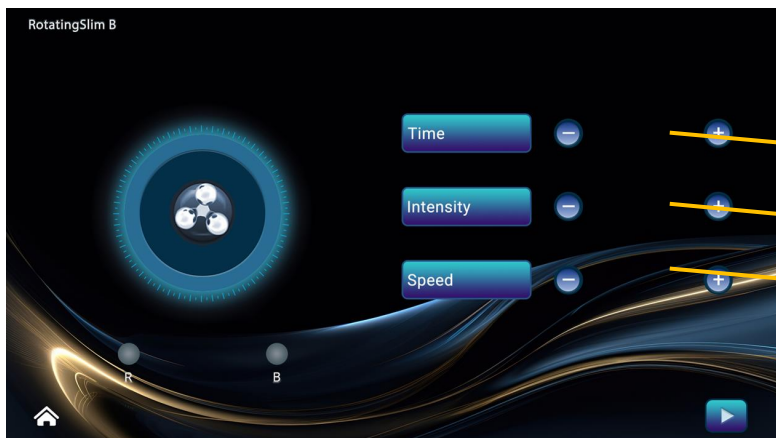

15–20

5–8

5–8


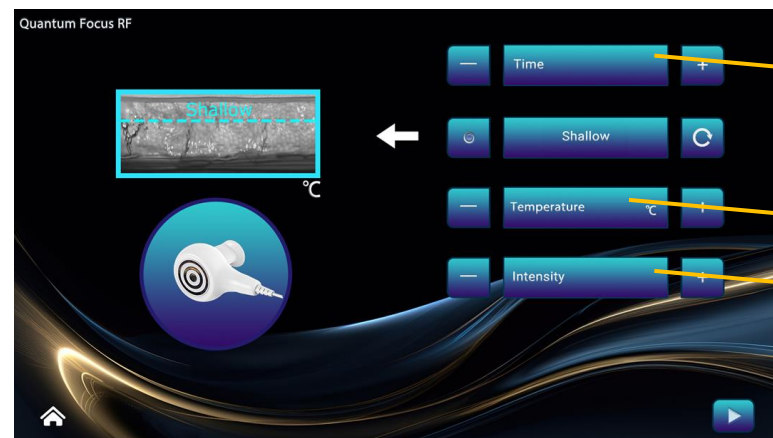


15–20

Start with 42

3–6

On this page, all parameters are pre-set in the system. Suggestions will be provided based on the selected area. Ensure that your client feels a strong sensation during the treatment but no pain.

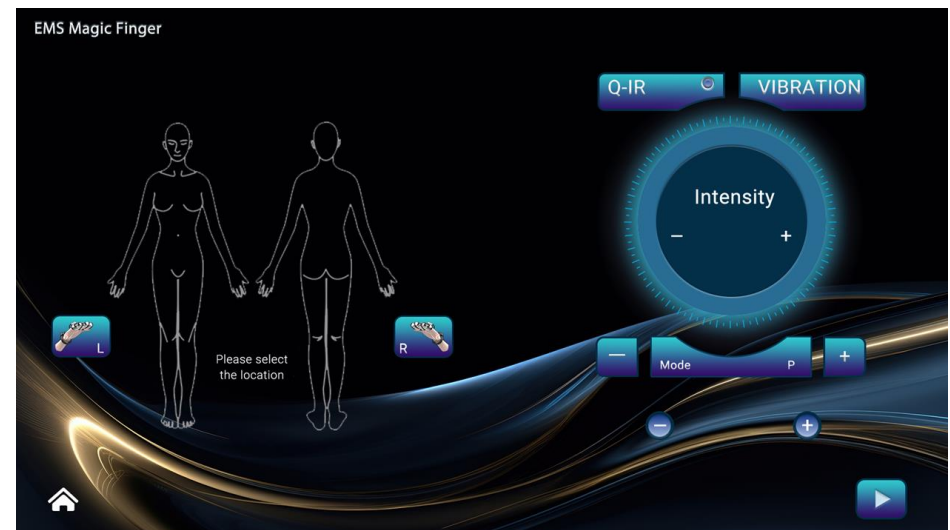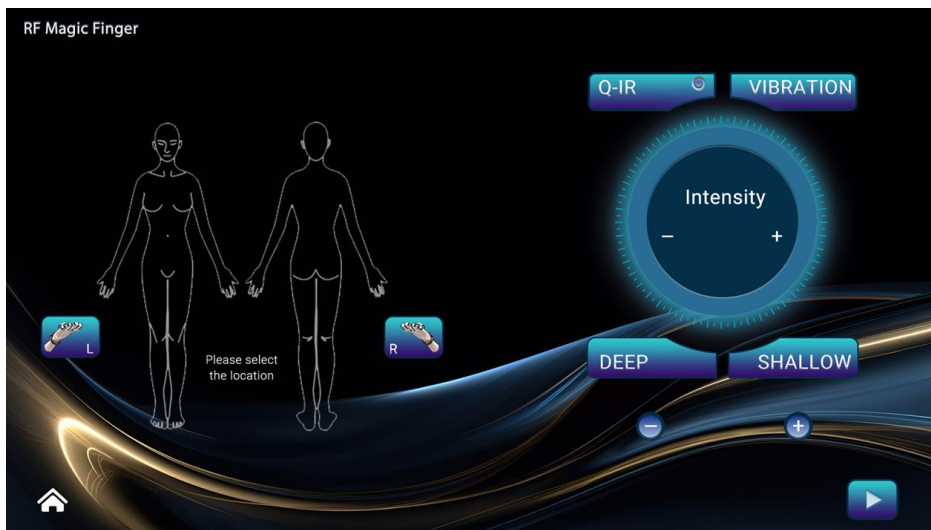

Specification

Model	LIYASHAPE - LS15
Input Power	AC110V/220V, 50/60Hz
Vacuum Intensity	10-80 Kpa
Rotation Angle	10-90°
Display	15.6 inches
LED Light	Red / Blue / Purple
Far Infrared Light	740—850 nm
LED cold light (red light)	490—650 nm
Dimensions	63*62*152 cm (Air shipping case)
Weights	72kg(N/T)

Contact

We value your feedback and would appreciate any comments or suggestions on how we can improve this manual.

Please direct any feedback to [info@winkonlaser.com].