



WORLD AESTHETIX

User Manual
LED Photodynamic Therapy Be

**Statement**

Thanks for using the LED Photodynamic Therapy Bed independently developed and produced by WORLD AESTHETIX.

This manual is a special instruction manual for LED Photodynamic Therapy Bed of WORLD AESTHETIX. Any company or people cannot copy and backup this manual by any forms (electronic, mechanical, photocopying, recording or other forms) if without the written permission.

This user manual is only for the operation instructions of the LED Photodynamic Therapy Bed. If it is used for the operation of other equipment, the company does not assume any responsibility.

Non- LED Photo dynamic Therapy Bed itself quality problems: such as the user's abuse, misuse, unauthorized disassembly, repair of the equipment, failure to operate and maintain the equipment in accordance with this manual, loss, etc., equipment failures are not included in the free maintenance.

Manual Instructions

This manual will provide you with relevant rules and precautions for the installation of LED Photodynamic Therapy Bed, daily operation instructions, after-sales service, etc. Please keep the manual by your side for quick search.

In order to use this series of led Photodynamic therapy machines correctly, give full play to the excellent performance of the product and ensure the safety of users and equipment, please read this manual carefully before using the NUSOL series led Photodynamic therapy machine. Improper use may cause abnormal operation of the led Photodynamic therapy machine, malfunction, shortened service life, and even equipment damage, personal injury or death!

This manual is written based on the existing functions and configurations of the product. If new functions and configurations are added, this manual will be revised accordingly, and the new version of the manual will be mailed to you along with the product.

If there is something unclear, you can contact our company in the following ways:

Company: WORLD AESTHETIX

Website: www.worldaesthetix.com



Content

1.Product Introduction	5
1.1Product Overview	5
1.2 Product Appearance	5
1.3Product Features	6
1.4 Applicable Places	6
1.5 Working Principle.....	6
1.6 Product Efficacy	7
1.7 Unsuitable People	8
1.8 Precautions for Use	8
1.9Product Parameters	10
2.Product Installation	11
2.1 Unpacking Inspection	11
2.2 Product Installation Specifications	11
2.3 Product Safety requirements	12
2.4 Technical specification	13
2.5 Installation Requirements	13
2.5.4.1 Electrical Configuration Requirements	14
2.5.4.2 Leakage protector installation	15
2.6 Product Installation	16
3. Product Operation	17
3.1 Internal control system operation panel	17
3.2 Note for Machine use	23
4. Faults and troubleshooting	24
4.1 No Display after Power On	24
4.2 The power air switch trips after power-on	24
4.3 The display is normal after power on, but the lamp does not work	24
5.Maintenance and Inspection.....	24
5.1 Daily Maintenance	24
5.2 Regular Maintenance	25
5.3 Storage	25



6. Product Warranty	25
6.1 Warranty Description	25
6.2 Not Included In the Free Warranty Service	26
6.3 After-Sales Record Sheet.....	26
7. Contact Us	27



1. Product Introduction

We have always maintained product stability, fashion and durability, and through continuous R&D and innovation, we have enriched product features and functions to design more cost-effective products for customers. Let us look forward to your safe and comfortable experience together!

1.1 Product Overview

LED Photodynamic Therapy Bed adopts the design of European space-level full-cabin; the internal design has undergone many clinical trials and has developed a design that conforms to the human body. When the customer lies down, the whole body is wrapped and very comfortable. With the built-in music function, you can relax while enjoying the beauty

1.2 Product Appearance





1.3 Product Features

- (1) Use Taiwan LED light source, high effect, safe and stable;
- (2) Using touch screen control management system, intelligent control throughout the Timing, query, memory, remote control and other functions;
- (3) Equipped with a surround sound system, supporting smart Bluetooth & Wifi connection to play music;
- (4) Equipped with plasma fans, which can effectively ventilate and dissipate heat, greatly improving comfort.

1.4 Applicable Places

In nursing homes, clinic, hospitals, postpartum rehabilitation institutions, high-end clubs, gyms, beauty salons, sports club, health and wellness centers, etc., Mexican has a strong technical research and development team, which can also be customized according to your needs. Please consult us for details.

1.5 Working Principle

World Aesthetix Photodynamic Therapy Bed uses high-tech nanotechnology for pure physical light therapy. It uses special chip technology to obtain red visible light bands of 600~700nm. This band penetrates deeply into the human body and has better curative effect. Its biological effects are mainly photochemical. action, rather than thermal action.

The mechanism of action of health physiotherapy cabin is to produce photochemical effects on biological cells. It acts on the skin through broad-spectrum strengthening, promotes the generation of fibroblasts, engulfs pigment cells, slows down the growth of melanin, and decomposes the melanin deposited on the skin. As a result, the skin tone gradually becomes fairer, and the originally dull skin becomes transparent and shiny after being irradiated by red light; it activates the mitochondria and various enzymes in the human body, promotes the production of elastin and collagen, and makes the skin plump , firm and smooth; improve blood circulation, enhance the vitality of human cells, enhance metabolism, accelerate the elimination of sick or dead cells, promote the body's self-repair function and cell regeneration ability, and enhance human immunity and disease resistance.



1.6 Product Efficacy

- (1) Relieve cardiovascular and cerebrovascular diseases: photodynamic therapy has a warming effect on the skin and mucous membranes, dilates blood vessels in deeper tissues, improves blood circulation, purifies the blood, and can prevent or alleviate cardiovascular and cerebrovascular diseases such as myocardial infarction and brain infarction, coronary heart disease, hypertension, hyperlipidemia, hyperglycemia, etc.
- (2) Improve pain: It can increase the catalase activity of cell mitochondria, promote the solidification of diseased tissue proteins and the production of new squamous epithelial cells, accelerate the absorption of exudates, weaken muscle tension, and improve various nerves, tissue, and joint pain, dysmenorrhea, gout, sports injuries, and other pains.
- (3) Promote the dissipation of inflammation: it can promote local blood circulation, increase the phagocytic function of cells and macrophages, control infections, and promote the dissipation of inflammation, such as: chronic inflammation, such as neurodermatitis, eczema, chronic lymphadenitis, etc., and various causes of osteoarthritis, such as senile osteoarthritis, rheumatoid arthritis, etc.
- (4) Promote wound healing: It can improve blood circulation, promote wound regeneration and repair, to achieve pain relief, reduce swelling, promote granulation tissue growth, shorten wound healing time, reduce postoperative adhesions, promote scar softening, and reduce scar contracture.
- (5) Enhance physical fitness: It can increase the catalase activity of mitochondria, thereby increasing glycogen content, protein synthesis, and adenosine triphosphate decomposition, enhancing cell viability, and enhancing the body's self-repair function and cell regeneration ability.
- (6) Relieve acne: It can promote skin blood circulation, enhance the vitality of human cells, enhance metabolism, speed up the elimination of sick or dead cells, and can alleviate and fade facial acne (acne), freckles, scars, postpartum stretch marks, etc..
- (7) Skin lightening and spot-lightening: It acts on the skin through broad-spectrum strengthening and promotes the production of fibroblasts, so that the pigment cells are swallowed, so that the skin tone will gradually become fairer, and the originally dull skin will also become irradiated by the red light. Transparent, exudes a bright luster.
- (8) Wrinkle removal and skin rejuvenation: stimulate the production of collagen,



increase skin elasticity, smooth wrinkles, shrink pores, and make skin firm, smooth and delicate.

- (9) Improve sleep: It can adjust the autonomic nerves to maintain a good state, effectively improve or eliminate headaches, dizziness, cold limbs, etc., improve sleep, and improve sleep quality.

1.7 Unsuitable People

- (1) Pregnant women, customers with cardiac pacemakers, and patients with malignant tumors.
- (2) Customers who are allergic to light therapy or who take photosensitizers (such as chlorpromazine).
- (3) Customers with active hemorrhage or shock, and internal organ damage.
- (4) Customers with high fever, active tuberculosis, vasculitis obliterans, etc.
- (5) Patients with severe heart and kidney disease, such as heart failure, myocarditis, uremia, etc.
- (6) Patients with skin diseases should use it under the guidance of a doctor.

1.8 Precautions for Use

1.8.1 Usage method

- (1) Clean before use: Vitamin C exfoliates for better absorption

Dead skin on the skin will hinder the absorption of light waves, so before using the health physiotherapy cabin, it is necessary to exfoliate the body first, so that the skin can absorb the red light waves better and faster, and improve the beauty effect. Exfoliating products containing vitamin C are recommended for smoother skin with a better texture and feel after exposure to light.

- (2) Pre-use care: use essence milk to enhance the effect

Apply essence milk and hyaluronidase all over the body before irradiation. After being irradiated by the red light in the health physiotherapy cabin, the skin's ability to absorb essence will increase by 3 times, allowing the skin to double absorb essence and enhance the beauty effect.



(3) Course of treatment: continuous use, better effect

It is recommended to use it once every other day at the beginning, 2-3 times a week, and 12-15 times a month. After about 1 month, the naked eye can see the obvious effect. After continuous use for about 3 months, the skin is in the best condition. After that, it needs to be illuminated 1-2 times a month to make the effect more durable.

Once every 20 minutes: fatigue disappears, physical strength recovers, and the mood is happy. When taking a bath that day, the skin of the whole body will feel smooth and tender, and the blood viscosity will decrease.

12 times in 30 days: smooth bowel movements, improved sleep, disappearance of bags and dark circles under the eyes, ruddy and shiny complexion, smooth and delicate skin all over the body.

22 times in 45 days: the wrinkles on the face and the whole body are gradually smoothed, and the spots on the face are gradually faded.

30 times in 60 days: Forehead wrinkles and crow's feet disappear, buttocks are lifted and abdomen tightened, and the skin of the whole body is tightened.

40 times in 90 days: Blood lipids, blood pressure, blood sugar decreased, joint pain disappeared, and wounds healed faster.

Long-term: Improving immunity and disease resistance, chronic diseases are under control, skin is firm and traceless, and aging is delayed.

1.8.2 Precautions

(1) It cannot be used to treat congenital skin hypoplasia, such as warts, spots, moles, etc.;

(2) If you are taking drugs recently, make sure that the drugs you are taking are not "photosensitive" drugs, which will produce chemical reactions under light;

(3) Before use, make-up must be removed, all accessories on the body must be removed, and a nude photo should be taken (note: sensitive parts of the skin should be covered with towels or clothing);

(4) Pay attention to eye protection during use, please remove contact lenses during use, and must wear special eye protection (eye patch);

(5) Do not take a bath within two hours after use;

(6) After the first use, observe whether there is any red light allergic reaction, if not, continue to use according to the course of treatment.



1.9 Product Parameters

Product Model	M6N
Light Source	Taiwan 0 2W LED light source
LED Exposure Angle	120°
LED Light Panels Size	2000x80mm
Led Bead QTY	41600 Pcs
Max Output Power Designed	6500W
Irradiance	Around 50 mw/cm ² (The exact Irradiance varies with the choice of the different wavelength LED) ,at LED 129mw/cm ²
Power Supply	Constant current source power supply
Current (220V)	35 A
Frequency	1-10000Hz
Duty Ratio	1-100%
External Control	Wireless Pad control system
Exposure Angle	360 degrees full tank
Stereo System	Standard Configuration
Bluetooth	Standard Configuration
Active Cooling System	Standard Configuration
Body Cooling System	Optional Configuration
Product Size	L2198 * W1157* H1079mm / cabin height: 430mm
Net Weight	300KG (Reference Value)
Bearing Weight Limit	300KG (Reference Value)
Voltage	220V 380V
Color Available	Grey Pearl White
Gross Weight	350KG
Packing Size	228*121*154cm (Reference Value)
Market Time	2022



2. Product Installation

2.1 Unpacking Inspection

Please check if the packaging is intact before unpacking. After unpacking, take out the parts, please check the following items:

- (1) Confirm that there is no damage to the machine during transportation, and check that the appearance of the machine is not damaged or notched;
- (2) Confirm that the packing box has an instruction manual and a warranty card (the warranty card is on the last page of this manual);
- (3) Check whether the machine nameplate information is consistent with the ordered product;
- (4) If you have ordered accessories, confirm that the received accessories are what you need.

Warning: Do not install or operate any LED Photodynamic machine that is damaged or has faulty parts, otherwise there is a danger of injury. If you find any questions, please get in touch with Merican and solve them immediately.

2.2 Product Installation Specifications

- (1) The machine must be grounded correctly before using. When multiple machines are grounded, it is recommended that you do not use a common ground wire to avoid the ground wire forming a loop;
- (2) In order to prolong the service life of the lamp and the machine, please turn off the power in time when not in use;
- (3) After the main power switch is turned off, you must wait at least 3 minutes for the machine to cool down completely before allowing installation/ dis-assembly operations;
- (4) It is forbidden to install and debug under non-insulated conditions to avoid electric shock;
- (5) The indoor air must be kept unblocked, and the distance between the outer edge of the machine and the wall is not less than 30cm;
- (6) Please refer to the description of this manual for the installation and use environment



of this machine. Failure to meet the environmental requirements may reduce the service life of the machine or even cause damage.

2.3 Product Safety requirements

- (1) The safety and correctness of the machine can only be guaranteed when it is connected with the machine provided by the company, and the safety of the equipment provided by any other company cannot be guaranteed;
- (2) In order to prevent circuit danger, the machine can only be connected to a power socket with protective grounding. If the power socket is not connected to a grounding wire, or the integrity of the protective grounding wire is questioned, please do not use the socket;
- (3) When a sudden power failure occurs during the normal use of the machine, the machine will not cause damage to the operator or user. Please turn off the power switch first, and perform normal operation according to the instructions when the power is turned on again;
- (4) Do not open the casing of the machine by yourself; otherwise there may be a risk of electric shock. The maintenance or upgrade of the machine must be carried out by maintenance personnel trained or authorized by the company;
- (5) In order to prevent electric shock and reduce machine failure, the machine must not be flooded. If the machine is accidentally exposed to water, please stop using it immediately and hand it over to professional technicians before using it again;
- (6) Before using the machine, check the machine and its connecting cables and accessories carefully to ensure that they can work normally and safely;
- (7) Use according to the parameters specified by World aesthetix;
- (8) In order to ensure the machine run under safe, all kinds of replaceable parts, accessories and various consumables equipped with the machine should use the products provided or designated by World Aesthetix;
- (9) Use alcohol with a concentration of 95% to clean and keep the surface of the machine clean. Wipe with alcohol once a week at least.
- (10) If the wire is damaged, it must be replaced by the service agent or related professionals to avoid injury;
- (11) When the machine is not in use, the machine door should remain normally open to



extend the service life of the machine's mechanical devices.

2.4 Technical specification

Project		Description
Control Characteristics	Start time	<30 seconds
	Operation control	Wireless Pad control system
	Modulation method	LED Touch screen
	Temperature	0°C~70°C
Ambient	Environmental humidity	≦75%RH, without condensation
	Degree of protection	ip 53
Structure	Cooling method	Air Cooling
	Installation method	Vertical installation

2.5 Installation Requirements

This machine needs to be installed indoors. During installation, there is no obstruction within 1 meter around the machine to ensure normal heat dissipation.

2.5.1 Temperature and Humidity

- (1) The ambient temperature must be between 0°C and 70°C;
- (2) The relative humidity of the air is less than or equal to 75%RH, without condensation.

2.5.2 Other Installation Environment Requirements

- (1) Install in a place that is unlikely to be subjected to severe vibration and impact;
- (2) Install in a place where metal powder, dust, oil, water, etc. cannot penetrate into the machine;
- (3) Avoid installing in places with oil mist, steam, and salt;
- (4) Avoid installing in places directly exposed to the sun;
- (5) Avoid installing in dusty and metal powder places;



- (6) It is strictly forbidden to install in places with flammable and explosive materials such as anesthetics.

Note: There is special installation environment requirements, please consult and confirm in advance.

2.5.3 Installation Interval and Distance Requirements

- (1) The distance between the four sides of a machine and the wall (30cm).
- (2) The distance between two machines (100cm).


The machine must be equipped with leakage protection/air switch; otherwise there is a danger of injury. This device is not provided by our company, please configure by yourself after consulting.

2.5.4 Electrical Configuration Requirements & Installation


The LED Photodynamic Therapy Bed must be equipped with a leakage protection switch or an air switch; otherwise there is a danger of injury. This device is not provided by our company. please configure by yourself after consulting. The power cord of the device must be directly connected to the junction box of the dedicated circuit (it is **STRONG RECOMMENDED** to be connected by a professional electrician, refer to the power configuration of the local and/or country code, and be equipped with appropriate circuit breakers and wire gauges).

2.5.4.1 Electrical Configuration Requirements

- (1) If the power supply is configured with 220V: It is necessary to configure a dedicated single-phase circuit capable of providing 40amp at the installation location (see Figure 1 below).
- (2) If the power supply is configured with 380V: It is necessary to configure a dedicated three-phase circuit capable of providing 3*25amp at the installation location (see Figure 2 below).

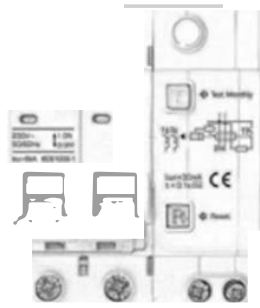

(Figure 1: 22V,40amp)

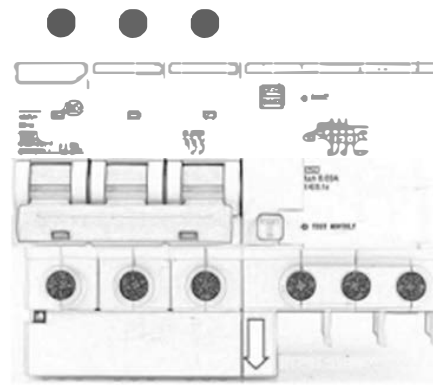


(Figure 2: 380V, 3*25amp)

2.5.4.2 Leakage protector installation

The installation of such large electrical equipment must use a dedicated circuit directly connected to the wiring box. We recommend that a professional electrician or the sales representative of the machine be contacted to confirm the installation requirements. Refer to local and/or national regulations for appropriate power supply and wiring tables. **The socket must be grounded.**

(1) Single-Phase Leakage Protector

There are a total of three wires; the Neutral Wire (blue), the Live Wire (brown), and the Ground Wire (yellow-green) are connected to the corresponding interfaces as shown in the figure below.

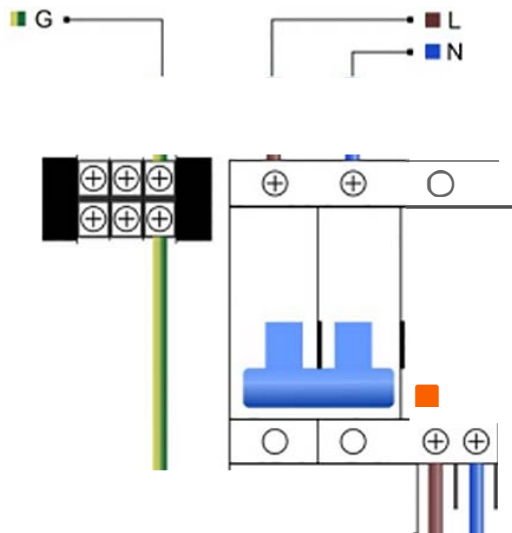


(2) Three-Phase Leakage Protector

There are a total of five wires, the Neutral Wire (blue), the Live Wire (brown, gray,



black), and the Ground Wire (yellow-green) are connected to the corresponding interfaces as shown in the figure below.

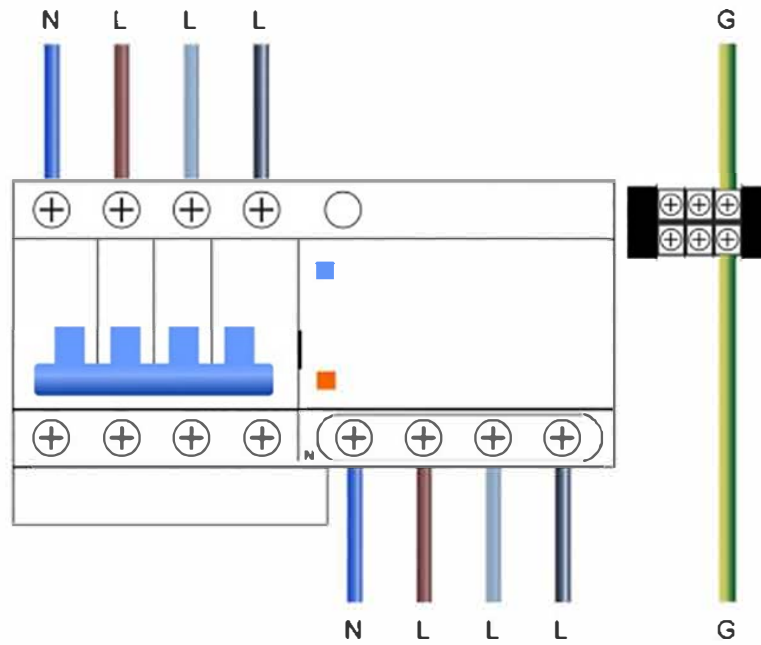

2.6 Product Installation





2.6.1 Disassemble Instruction



1. Please Unplug the wires connection between Upper canopy and middle part.
2. Please completely open up the upper canopy to raise canopy up a highest point ,and arrange 4 people hold the canopy 4 angles . then remove the screws on both back brackets No.: 1,2,3,4,5,6. (marked as picture)
3. After remove the screws , can take off the upper canopy from machine frame.

3. Product Operation

Please read the operating instructions carefully before using the M6N series of Photobiomodulation therapy bed, and operate in strict accordance with the operating instructions.

3.1 Internal control system operation panel

(1) Home Page

After connects with electricity for the device , control screen panel will enter the home page. As showed below .




The function description of each number marked on the page is as follows:

No	Button Icons	Function	Description
1,2		Time setting(Minute)	Press one time , add /Minus1 minute. Under working it cannot be adjusted.
3,4		Time setting(Second)	Press one time , add /Minus1 minute . Under working it cannot be adjusted
5		Time Display Area	When adjusting the time , it displays the current working time setting ; when under working , it displays the remaining working time (counting down)
6		Body Fan Cooling Level	Turn off fan : fan icon button display no number . Fan cooling level 1: Icon displays 1, mean level 1, weakest Fan cooling level 3: Icon displays 3,mean level 3, strongest When fan is working , the fan icon is rotating ,and number displays current working fan level.
7		Start /Pause/Continue	Start: Press one time , start working ; Pause: During working ,long press the button 2 seconds will pause the device working ; Continue : After pause, press again the button, will restart working.
8		Music Player Setting / Setting	1) Quick press one time the setting button , will enter the music setting page , After link with phone bluetooth, can play music via device speakers. If bluetooth disconnects, the icon of button is grey color ; if connects successfully, it is blue icon; 2) Long press 3 seconds the button, will enter the setting page .

(2) Music Player Setting Page

Music setting homepage as below :

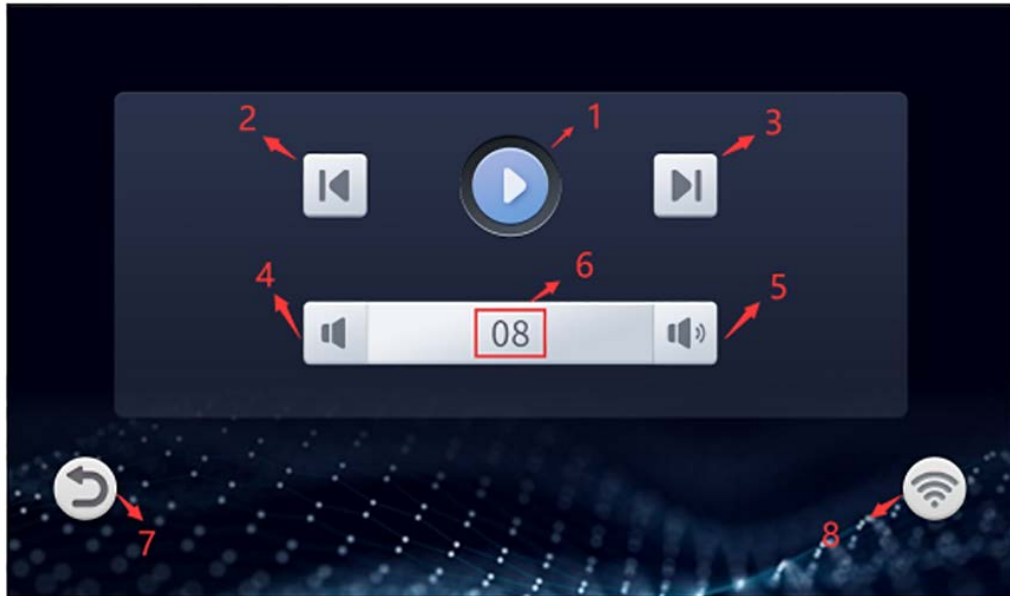

The function description of each number marked on the page is as follows:

No.	Button icon	Function
1		Music Play Pause Button
2		Last Song
3		Next Song
4		Volume -
5		Volume +
6		Current Volume level
7		Back to main homepage
8		Wireless Wifi Connection signal


The wireless connection icon has the following 7 status :



Status	Icon	Description
1		Wifi Disconnection.
2		Connected with the network, but didn't connect with network server, signal is bad
3		Connected with the network, but didn't connect with network server, signal is so so .
4		Connected with the network, but didn't connect with network server, signal is good.
5		Connected with the network and also network server, signal is bad
6		Connected with the network and also network server, signal is so so .
7		Connected with the network and also network server, signal is good.

(3) Pulse Setting Page

The pulse setting functional button is shown as below :

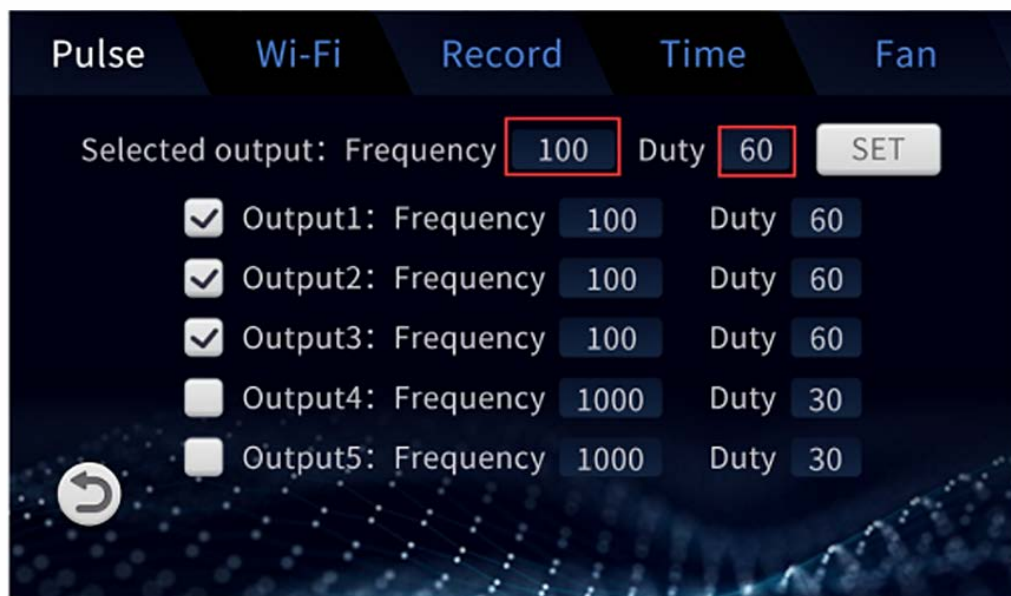

You can configure the frequency (Frequency), duty cycle (duty) and whether to output (check box) of each output separately, or you can check multiple outputs and fill in the frequency to be configured in the red box in the above figure (Frequency), duty cycle (duty), press the "SET" button to configure the filled-in frequency (Frequency), duty cycle (duty)



values for multiple selected outputs at the same time.

Need pulse : adjust figures on main frequency and Duty,press (set)to confirm for selected outputs .

No need pulse: put frequency 100 and duty 100 ,then press set to confirm for selected outputs . then restart device lights. Will be no pulsing.

(4) WiFi Setting



This page has 2 funtions :

1. WiFi configuration. Click the <Net name> edit box, and on the AP scanning page that pops up, select the name of the network to be connected, and enter the corresponding password in the Password edit box, and press START to start reconfiguring the WiFi network. The system will save the corresponding passwords of the 9 most recently connected network names. If the password edit box is not empty after selecting the network name, it means that the network has already been connected. At this time, there is no need to enter the password, and you can directly press the START button to configure the WiFi network.


2. Local configuration. You can modify the device name in the Device name edit box, and modify the local UDP broadcast receiving port in the Local port edit box. The port must be consistent with the UDP broadcast port set on the APP to connect to the corresponding APP.

(5) Records Setting




In this page, you can check recent 999 times using records .Press Last to check last using record, press next to check next using records.

(6) Time Clock Setting

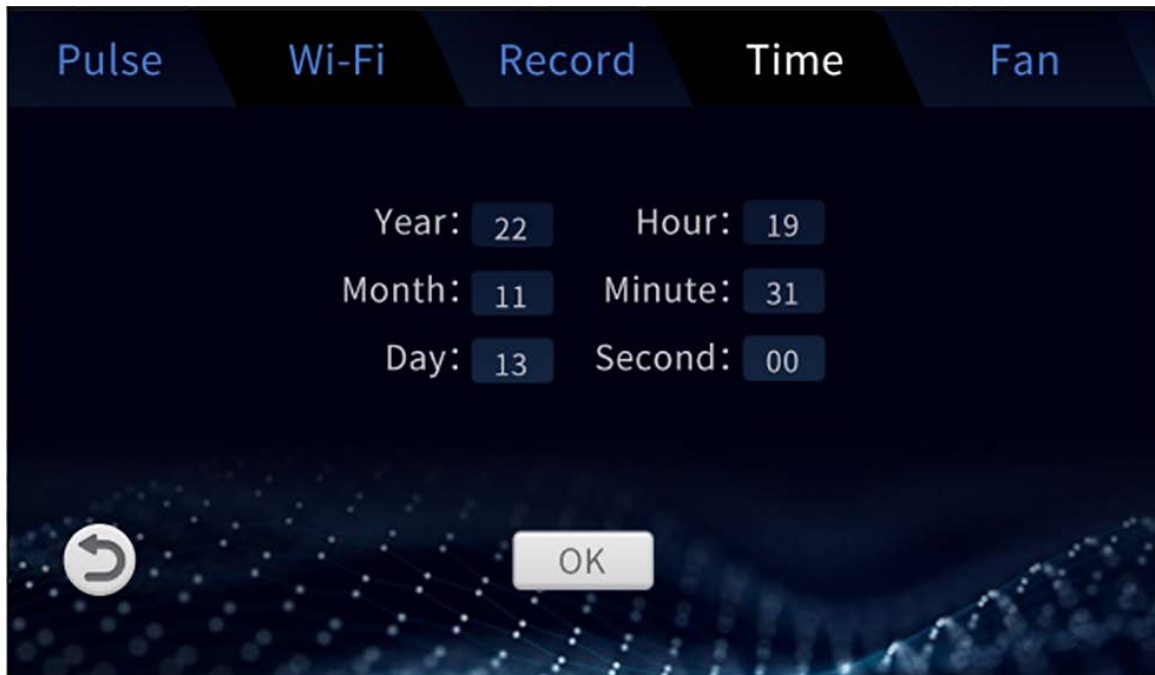

Input the correct time , press OK button to confirm and save time setting,press back button to return homepage. If no click OK, press back button to return homepage and won't save the records.

(7) Cooling Fan setting

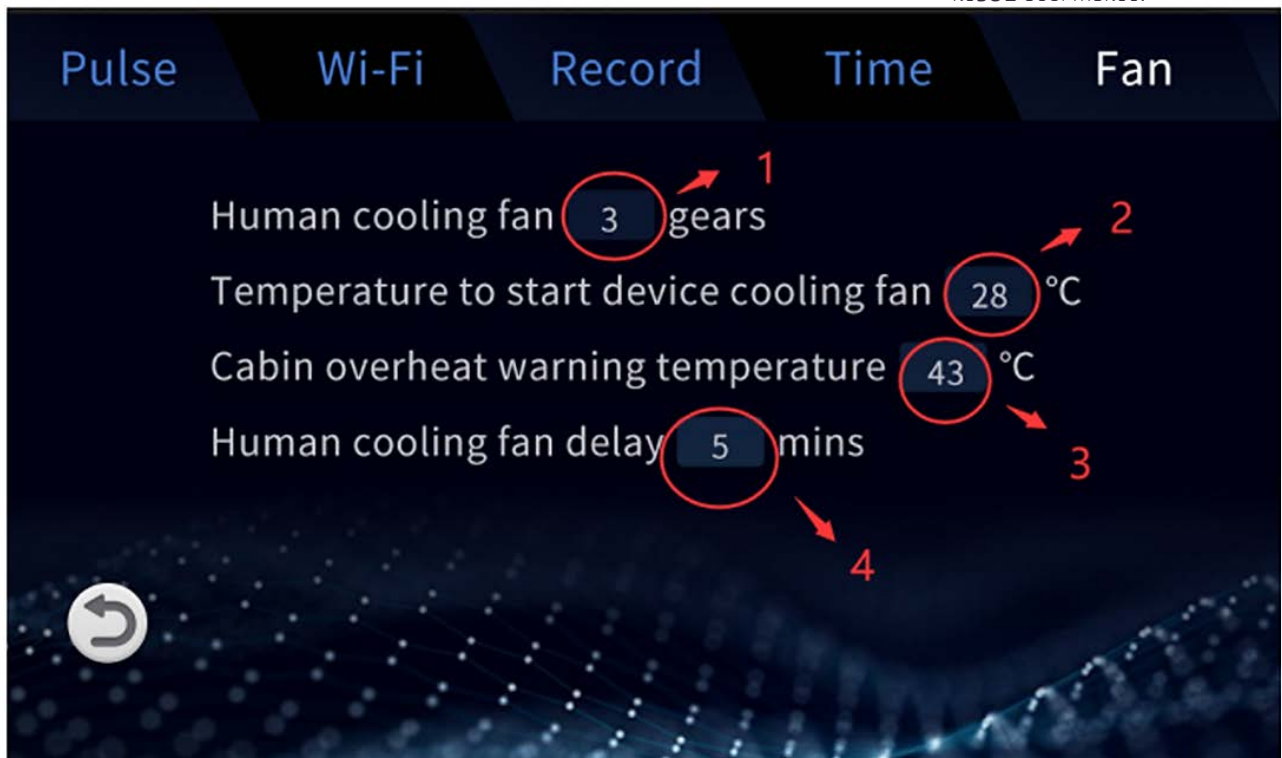

The function of each number marked component on the page is as follows:

No1. Body cooling fan level after session.

No2. Temperature to start device small cooling fans on canopy and back of cabin.

No.3 Device overheat warning temperature setting. When device heat is over this temperature, will make warning sound .

No4. Body Cooling fan delay time after session . If you set 5 minutes, when session is off, machine body cooling fan will automatically work 5 minutes to cool down device then auto-off .

3.2 Note for Machine use

- (1) When the machine is not in use, the machine door should be kept open and the plug should be unplugged;
- (2) When maintaining or cleaning the interior of the operation cabin, ensure that the power supply is disconnected, and do not use liquid to clean the live parts;
- (3) If the wire is damaged, it must be replaced by the manufacturer/service agent/related professionals to avoid injury;
- (4) The organic glass plate in the engine room is damaged, cracked or scratched, please replace it immediately;
- (5) After each use, use alcohol with a concentration of more than 95% to clean the transparent plexiglass in the lower warehouse; clean the whole machine once a week;



- (6) When the machine is in the standby state, it is strictly forbidden to touch or lie on the instrument with water, and it is strictly prohibited to clean the machine stains in the standby state.

4. Faults and troubleshooting

The following failures may be encountered during use, please refer to the following methods for simple failure analysis:

4.1 No Display after Power On

Use a multimeter to check whether the input power voltage of the solarium is normal.

4.2 The power air switch trips after power-on

Check whether there is a grounding or short circuit between the input power supplies, and eliminate the problem.

4.3 The display is normal after power on, but the lamp does not work

- (1) Check whether the tube interfaces is loose;
- (2) Check whether the tube is burned out, if so, please seeks service.

5. Maintenance and Inspection

5.1 Daily Maintenance

In order to prevent the solarium tanning machine from malfunctioning, to ensure the normal operation of the equipment, and to extend the service life of the equipment, daily maintenance of the solarium tanning machine is required. The contents of the daily maintenance are as follows:

Check Item	Content
Temperature	Confirm the ambient temperature is between 0°C and 70°C
Humidity	Confirm that the relative humidity of the environment is $\leq 75\%RH$ and there is no condensation
Oil mist and dust	Check that there is no oil mist, dust, or condensation in the machine
Machine	Check whether the machine has abnormal heat and vibration
Fan	Confirm that the fan is running normally and no debris is stuck, etc.
Input power	Check whether the input power voltage is normal



5.2 Regular Maintenance

In order to prevent the solarium tanning machine from malfunctioning and ensure its long-term high-performance and stable operation, the user must check the machine regularly (within half a year). The inspection contents are as follows:

Check item	Check content	Method of exclusion
Nut Screw Interface	Is The Nut Loose?	Tighten
Body	Dust, Dirt	Wipe Clean With 95% Alcohol
Fan	Abnormal Noise And Vibration	1) Remove Debris 2) Replace The Fan
Light Tube	Is The Light Tube Loose?	The Light Logo Is Facing The Outside
Air Inlet/Air Inlet	Dust And Dirt	Use Dry Compressed Air To Completely Remove Debris
Light Tube	Light Tube Does Not Light	Switch The Normal Light Tube And The Light Tube To Check The Status Of The Light Tube
The Equipment Door	Whether It Is Aligned	Adjust The Hinged Door So That The Two Are Precisely Connected
Wires And Their Interfaces	Whether It Is Damaged Or Loose	Contact Us To Guide Replacement

5.3 Storage

The Phototherapy machine must be placed in its packaging box before installation. If the machine is not in use temporarily, in order to make the machine meet the company's warranty and future maintenance, please refer to the following precautions for storage:

- (1) The storage must be placed in a dust-free, dry environment;
- (2) The ambient temperature of the storage location must be within the range of 0°C~70°C;
- (3) The relative humidity of the storage location must be 0%~75%RH, and there should be no condensation;
- (4) The storage location should avoid corrosive gases and liquids;
- (5) There should be proper packaging protection, and should be stored above the ground.

6. Product Warranty

6.1 Warranty Description

- (1) During the warranty period, for any faults caused by product quality, the Merican after-sales service center will repair the products or replace parts free of charge. Users are not required to disassemble and repair without permission;



- (2) Since the date of receipt of the machine, a one-year warranty for the whole machine/accessories will be provided, and the accessories will be replaced free of charge within one year; after the warranty period, life-long maintenance (repair fees will be charged according to the regulations), otherwise agreed in accordance with the agreement;
- (3) If users have special requirements for services, they can consult with the after-sales service center of Merican;
- (4) The relevant technical data of this machine can be provided to the technical service personnel authorized by our company when necessary.

6.2 Not Included In the Free Warranty Service

- (1) More than one year from the date of receipt of the machine;
- (2) Failure caused by unauthorized disassembly or modification of the machine;
- (3) Damage caused by non-professional installation, maintenance and assembly;
- (4) Damage caused by using non-original parts or inferior parts;
- (5) Failure caused by accidental beating or falling during use and handling;
- (6) Failure caused by lack of reasonable maintenance or failing to meet environmental requirements;
- (7) Failure caused by incorrect operation not in accordance with the requirements of the instruction manual;
- (8) Damage to machines and accessories caused by man-made causes;
- (9) Failure caused by self-maintenance without our company's permission;
- (10) Machine failure caused by unstable local voltage;
- (11) Failure or damage caused by force majeure such as fire and earthquake.

6.3 After-Sales Record Sheet

For products not covered by the warranty, our company provides paid maintenance services, and the same performance problem will enjoy a free warranty period of 3 months from the date of repair.

Product Number		
Sale Date		
User Profile	User Name	
	Mailing Address	
	Contact Number	
	E-Mail	



Repair Date	Repair Record	Remarks

7. Contact Us

If you need to order products or have any questions, you can contact us in the following ways:

Company: World Aesthetix

Website: www.worldaesthetix.com